




## LAYERING TECHNIQUE

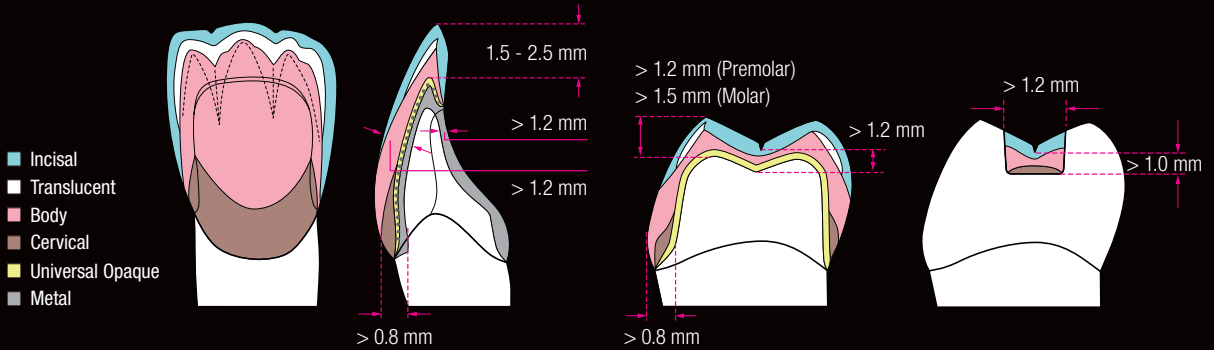

## ENAMEL COLOUR IMAGE

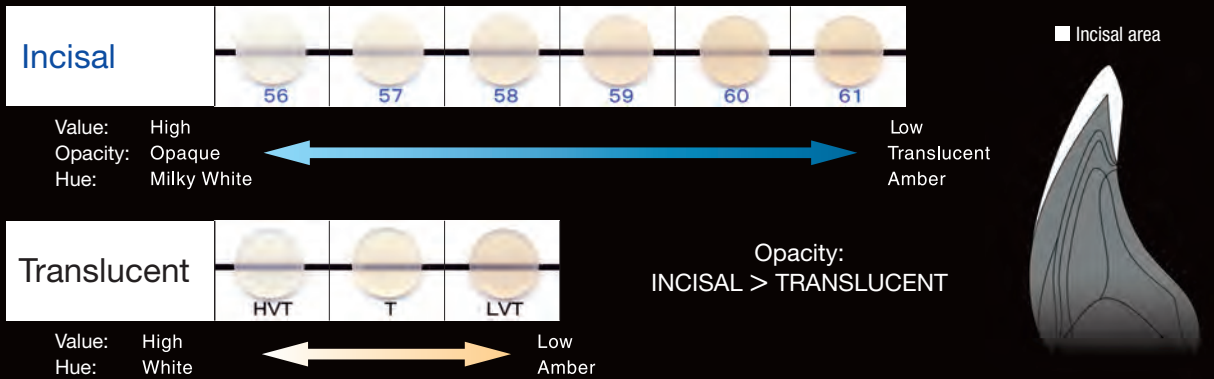

## LAYERING TECHNIQUE FOR GINGIVAL REPRODUCTION

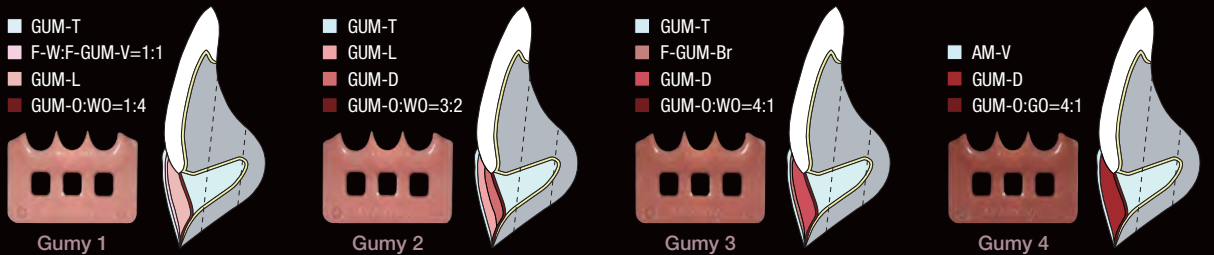

## LIGHT-CURING TIME

		Solidilite V	Solidilite EX
SHOFU Universal Opaque	Pre-Opaque	60	60
	Opaque	180	180
Ceramage Paste / Flowable (Pre-light-cure)		60	60
Final		180	300

(Seconds)

COLOUR	W0	W1	W2	W3	A1	A2	A3	A3.5	A4	rootA
UNIVERSAL OPAQUE	W0	W20	W30	A10	A20	A30	A3.50	A40	rootA0	
CERVICAL						AC1		AC2		
OPAQUE DENTIN					ODA1	ODA2	ODA3	ODA3.5	ODA4	ODrootA
BODY	W0B	W1B	W2B	W3B	A1B	A2B	A3B	A3.5B	A4B	rootAB
INCISAL		56		57	58		59			60

COLOUR	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4
UNIVERSAL OPAQUE	B10	B20	B30	B40	C10	C20	C30	C40	D20	D30	D40
CERVICAL		BC1		BC2		CC1		CC2	DC1		DC2
OPAQUE DENTIN	ODR1	ODR2	ODR3	ODR4	ODC1	ODC2	ODC3	ODC4	ODD2	ODD3	ODD4
BODY	B1B	B2B	B3B	B4B	C1B	C2B	C3B	C4B	D2B	D3B	D4B
INCISAL	57	58	59	60	58	59	60	59	60	59	

The printed colours might be different from the actual colours in this system.

COLOUR	R2	R3	R3.5
UNIVERSAL OPAQUE	R20	R30	R3.50
OPAQUE DENTIN	ODR2	ODR3	ODR3.5
BODY	R2B	R3B	R3.5B
INCISAL	58	59	

UNIVERSAL OPAQUE	MO	BGO	GO	WO	GUM-O	IRO	VpO	VO	PO	PrnO	
TRANSLUCENT	T	HVT	LVT	T-Glass	BG	GT	CT-A	CT-B	CT-R	GUM-T	
INCISAL	56	57	58	59	60	61					
CONCENTRATE	MI	WE	OC	AM-Y	AM-R	AM-V	MY	MP	GUM-L	GUM-O	GUM-Or
FLOWABLE COMPOSITE RESIN	F-T-Glass	F-59	F-ODA3	F-A3B	F-rootAB	F-Br	F-W	F-CT-A	F-BT		
	F-GUM-Br	F-GUM-V	F-GUM-R								